



Alcohol-based, rapid disinfection of surfaces\*.

**New  
formula**

## kodan® (N) wipes

### Our Plus

- broad efficacy within very short time
- residue-free
- ready to use

### Application areas

The product is suitable for disinfecting surfaces.

### Instructions for use

Thoroughly wipe the surface with the cloth and allow the active ingredient solution taking effect. Ensure complete wetting and maintain the surface moist during the entire exposure time. Ensure that all visible dirt is removed before disinfection. Treat only small areas. The wiping area depends on the room temperature and the structure of the surfaces to be disinfected. Pay attention to hygienic aspects when opening and threading the first cloth. Make sure to close the lid after use.

#### Refilling the box

Disinfect your hands first and put on a fresh pair of protective gloves and a gown. Ensure that the gloves are not contaminated by other activities during the filling process. Transfer the sticker with the expiry date to the dispenser box. Unscrew the lid, place the refill bag completely in the dispenser box (if necessary, shape it to fit beforehand) and cut off the edge of the bag. Pull the pre-plucked cloth from the middle of the roll, thread it from below into the guide slot provided in the cover and pull it through. Disinfect the lid and screw it on. For high-risk areas, schülke recommends not using refill bags. For professional use only.

### Microbiological efficacy

Efficacy	Concentration	Contact time
<b>bactericidal</b> EN13727, EN16615 - dirty conditions	ready-to-use	1 min.
<b>bactericidal</b> - dirty conditions	ready-to-use	1 min.
<b>tuberculocidal</b> EN14348 - dirty conditions	ready-to-use	1 min.
<b>yeastocidal</b> EN13624 - dirty conditions	ready-to-use	1 min.
<b>yeastocidal</b>	ready-to-use	1 min.
<b>yeastocidal</b> EN16615 - dirty conditions	ready-to-use	1 min.
<b>fungicidal</b> EN13624 - dirty conditions	ready-to-use	2 min.
<b>virucidal against enveloped viruses</b> in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	30 sec.
<b>Norovirus</b> EN14476	ready-to-use	1 min.
<b>Rotavirus</b> in accordance with DVV	ready-to-use	30 sec.
<b>Adenovirus (type 5)</b> in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	5 min.



# kodan<sup>®</sup> (N) wipes

## Product data

### Composition:

100 g solution contain the following active ingredients:

25 g Ethanol (94%w/w), 35 g Propan-1-ol

Labelling according to Regulation (EC) No. 648/2004:

perfumes

### Chemical-physical data

Color	colourless
Density	ca. 0,89 g/cm <sup>3</sup> / 20 °C / of the active solution
Flash point	27 °C / Method : DIN 51755 Part 1 / of the active solution
Form	Aqueous, alcohol containing solution on non-woven
pH	ca. 7 / 20 °C / of the active solution
Viscosity, dynamic	No data available

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: [www.schuelke.com](http://www.schuelke.com).

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: [info@schuelke.com](mailto:info@schuelke.com)

\* except for medical devices

## Special advice

### Use disinfectants safely. Always read the label and product information before use.

The product has good material compatibility with metals and plastics (except acrylic glass and alcohol-sensitive paints). Surfaces that are particularly sensitive to alcohol, such as acrylic glass, must not be treated. When using alcoholic disinfectants, please observe the fire and explosion protection in accordance with the trade association regulation "Disinfection Work in Health Care". Keep away from heat and sources of ignition. Wearing protective gloves is advised. Further information is available on request. If handled properly, schülke guarantees a shelf life of 3 months after the package has been opened.

## Information for order

Item	Delivery form	Item no.
kodan <sup>®</sup> (N) wipes refill (20 x 90 wipes)	20/Carton	on request
kodan <sup>®</sup> (N) wipes (10 x 90 wipes)	10/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Application aids

Application aids	Item no.
Wall mounting for the dispenser (small)	134417

## Related Products

- mikroqid<sup>®</sup> AF wipes
- mikroqid<sup>®</sup> sensitive wipes
- mikroqid<sup>®</sup> universal wipes premium



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)